



Designers and Manufacturers of Mobile Health Clinics

ADI Technical Bulletin #2

Do's & Don'ts

DO

1. Place the x-ray remote control in a safe place.
2. Secure all loose instruments, equipment and trays – countertops are to be clean and free of debris.
3. Secure all pocket doors, sliding doors, privacy curtains and cabinetry drawers.
4. Lock all filing cabinet drawers.
5. Strap down all mobile cabinets, lights, stools, and x-ray head(s).
6. Drain the ultrasonic cleaner and x-ray film processor of chemicals.
7. Secure the jacks, steps and awning for travel.
8. Securely latch the basement compartment doors.
9. Disconnect the shore power (land line) cord, water and/or waste hoses, and telephone / CAT6 lines.
10. Walk around the exterior of the van before driving away. Start at the driver's side windshield and walk in a counter clockwise direction.

DO NOT

1. **DO NOT** drive away before completing the “DO” list above.
2. **DO NOT** drive away before visually checking the jacks, steps and awning (if applicable) to ensure they are correctly stored.
3. **DO NOT** drive away with the shore power (land line) cord, water and/or waste hoses, or telephone / CAT6 lines connected.

ADI Mobile Health - FACTORY

19425 SW 89th Avenue
Tualatin, OR 97062 USA
www.adi-mobilehealth.com

Phone: +1-503-885-0886
Fax: +1-503-885-0417
tech@adi-mobilehealth.com
sales@adi-mobilehealth.com